


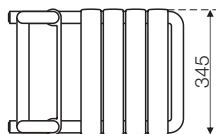
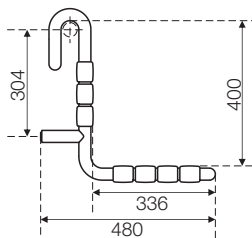
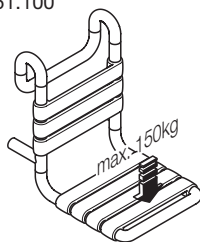
- Ⓓ Montageanleitung
- ⒼⒷ Mounting instructions
- Ⓕ Notice de montage
- ⒼⓁ Montage-instructie
- Ⓘ Istruzioni di montaggio
- Ⓔ Instrucciones de montaje
- ⒼⓁ Instrukcja montażu

<b>D</b> <b>Inhalt</b>		<b>NL</b> <b>Inhoud</b>		<b>E</b> <b>Contenido</b>	
Wichtige Hinweise	3	Belangrijke tips	6	Indicaciones importantes	8
Gebrauchsanleitung	10	Gebruiksaanwijzing	10	Instrucciones de uso	10
<b>GB</b> <b>Content</b>		<b>I</b> <b>Contenuto</b>		<b>PL</b> <b>Treść</b>	
Important information	4	Informazioni importanti	7	Ważne wskazówki	9
Instructions for use	10	Istruzioni per l'uso	10	Instrukcją obsługi	10
<b>F</b> <b>Sommaire</b>					
Remarques importantes	5				
Notice d'utilisation	10				




801.51.100

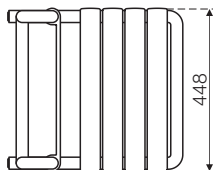
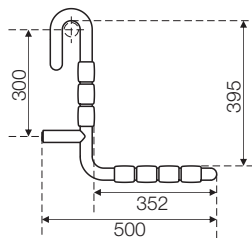
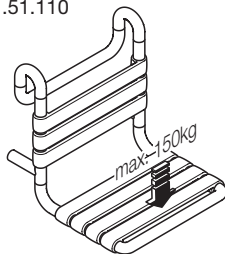
801.51.110

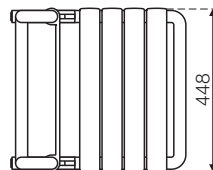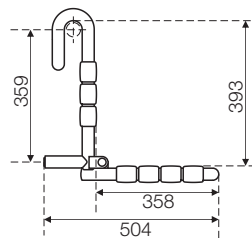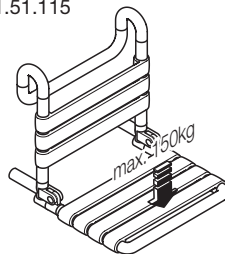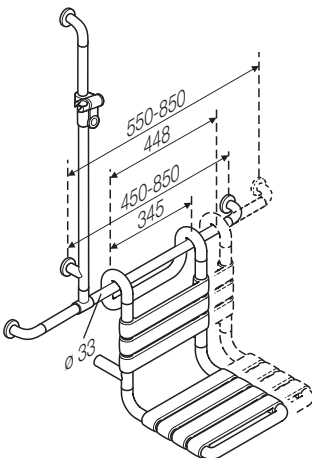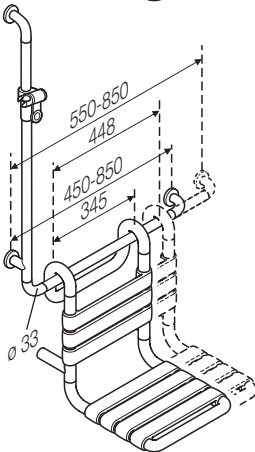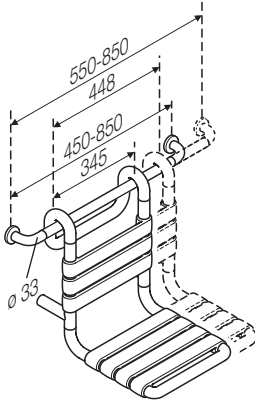801.51.115

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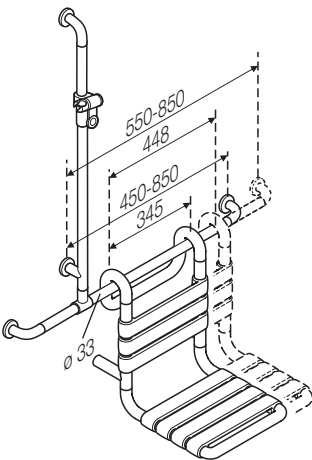
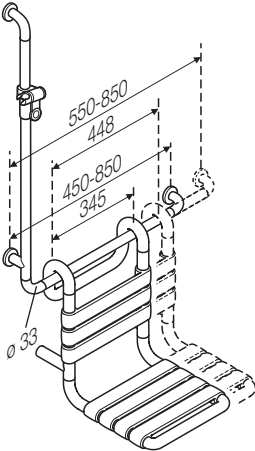
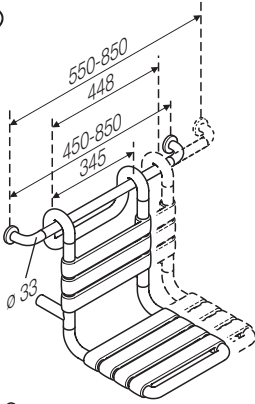
### Purpose

The HEWI suspended seat is a seat for people with and without physical disabilities. The HEWI suspended seat is designed for a maximum vertical static load of 150 kg.

### Mounting information

The following points must be observed:

- The suspended seat is suspended from the appropriate HEWI handrail (dia. 33 mm) from above. The support separation must not exceed max. 850 mm (horizontal bar).
- Ensure that the wall structure and the handrail fittings are able to permanently withstand the anticipated load. Lightweight wall constructions must have a suitable supporting framework.

### Warning and hazard notices

- Improper mounting or use can lead to accidents involving serious injuries.
- Do not misuse the suspended seat, e.g. as a climbing aid.
- A certain vertical incline is normal when the seat is under load. In the event of damage, the suspended seat may only continue to be used following inspection by a specialist company.

### Care and cleaning instructions

HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough. When using detergents, please observe the following:

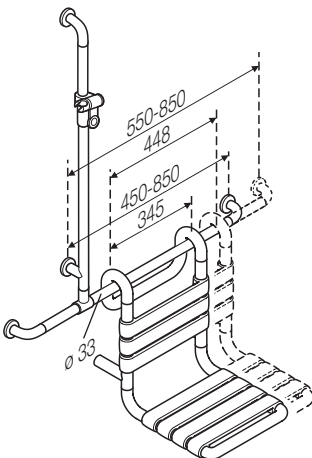
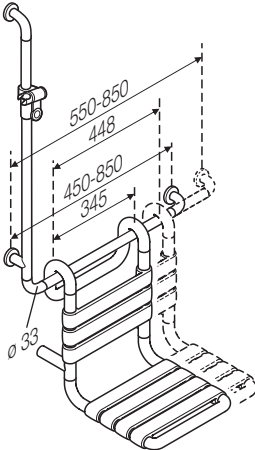
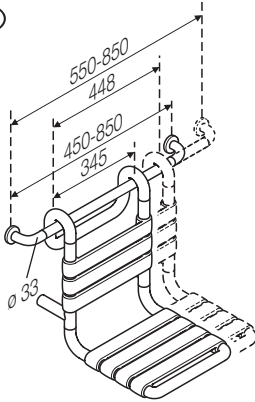
- Only use detergents with a pH of between 6 and 8.
- Detergents containing active substances such as phosphates, soap and tensides may be used.
- Follow the detergent manufacturer's instructions (concentration, contact time).
- Detergents containing acids, alkalis, bleach or scouring agents may not be used.
- Do not use scouring cleaning equipment, such as brushes

### Exclusion of liability

HEWI accepts no liability whatsoever for damage to persons or property that occurs or has occurred as a result of improper mounting or maintenance, particularly if the safety instructions and other points to note listed are not or have not been observed in full. HEWI accepts no liability whatsoever for damage occurring from incorrect cleaning.

**For further information, please contact HEWI Technical Support, tel. +44 1634 377688.**

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HEWI Heinrich Wilke GmbH